D	Kontaktloses Fieberthermometer Gebrauchsanleitung	2-17
GB	Non-contact fever thermometer Instructions for use	18-33
F	Thermomètre médical sans contact Mode d'emploi	34-49
E	Termómetro clínico sin contacto Instrucciones de uso	50-65
I	Termometro per febbre senza contatto Istruzioni per l'uso	66-81
TR	Temassız termometre Kullanım kılavuzu	82-97
RUS	Бесконтактный термометр Инструкция по применению	98-115
PL	Bezdotykowy termometr lekarski Instrukcja obsługi	116-131
CHN	非接触式体温计 说明书	132-146
Arab	ترمومتر قياس حرارة غير متلامس دليل الاستعمال	148-164

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Dear customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, gentle therapy, blood pressure/diagnosis, weight, massage and air.

Please read these instructions for use carefully and observe the information they contain.

With kind regards,

Your Beurer Team

1. Safety notes	19
2. Information about this thermometer	20
3. Unit description	22
4. Initial use	23
5. Switching on and setting the thermometer	24
6. Forehead measurement	26
7. Measuring object temperature/room temperature	28
8. Error messages	28
9. Batteries	30
10. Cleaning the device	30
11. Storing the device	31
12. Disposing of the device	31
13. Technical Data	31
14. Guidelines	33
15. Included in delivery	33

Caution!
Risk to the user.

Note!
Risk to the device.

Important information/tip

Cross-reference to another section.

1. Safety notes

1.1 Risks to the user

- Only use the device once you have read and understood these instructions for use.
- Retain these instructions for use. The instructions for use must be accessible to all users. All instructions must be followed.
- The thermometer needs to be in the room in which the measurement is taken for at least 30 minutes before use.
- The FT 90 thermometer is only designed for the measuring area on the human body stated in the instructions for use.
- The device is only intended for the purpose stated in these instructions for use.
- Children must not be allowed to use the device. Medical products are not toys.
- Check before each use that the lens is intact. If it is damaged, please contact your retailer or the service address.
- The device has been designed for practical use, but is not a substitute for a visit to the doctor.
- This device is not intended for commercial or clinical use.
- Should you have any questions about using the device, please contact either your retailer or Customer Services.

1.2 Risks to the device

- Do not subject the device to mechanical impacts.
- Do not expose the device to direct sunlight.
- Do not expose the device to liquids. The device is not waterproof. Avoid all direct contact with water or other liquids.
- Have the device repaired by authorised service centres only, otherwise its warranty is invalidated.
- Portable and mobile HF communication systems may interfere with this device. For more details, please contact our customer service at the address indicated or see the enclosed information sheet.

2. Information about this thermometer

The temperature measurement varies depending on the part of the body where the measurement is taken. In a healthy person, the variance can be between 0.2 °C and 1 °C (0.4 °F and 1.8 °F) in different parts of the body.

Normal temperature range with various thermometers

	<i>Measurements</i>	<i>Thermometer used</i>
Forehead temperature	35.8 °C to 37.6 °C (96.4 °F to 99.7 °F)	Forehead thermometer
Ear temperature	36.0 °C to 37.8 °C (96.8 °F to 100.0 °F)	Ear thermometer
Oral temperature	36.0 °C to 37.4 °C (96.8 °F to 99.3 °F)	Conventional thermometer
Rectal temperature	36.3 °C to 37.8 °C (97.3 °F to 100.0 °F)	Conventional thermometer

Beurer tip

- Temperatures measured with different thermometers should never be compared with one another.
- Tell your doctor what type of thermometer you used to take your temperature and on what part of the body. Also bear this in mind if you are diagnosing yourself.

Holding the thermometer for too long in the hand before taking a measurement can cause the device to warm up. This means the measurement could be incorrect.

Influences on body temperature

- A person's individual metabolism
- Age

Body temperature is higher in babies and toddlers than in adults. Greater temperature fluctuations occur faster and more often in children. Normal body temperature decreases with age.

- Clothing
- Outside temperature
- Time of day

Body temperature is lower in the morning and increases throughout the day towards evening.

- Activities

Physical and, to a lesser extent, mental activities increase body temperature.

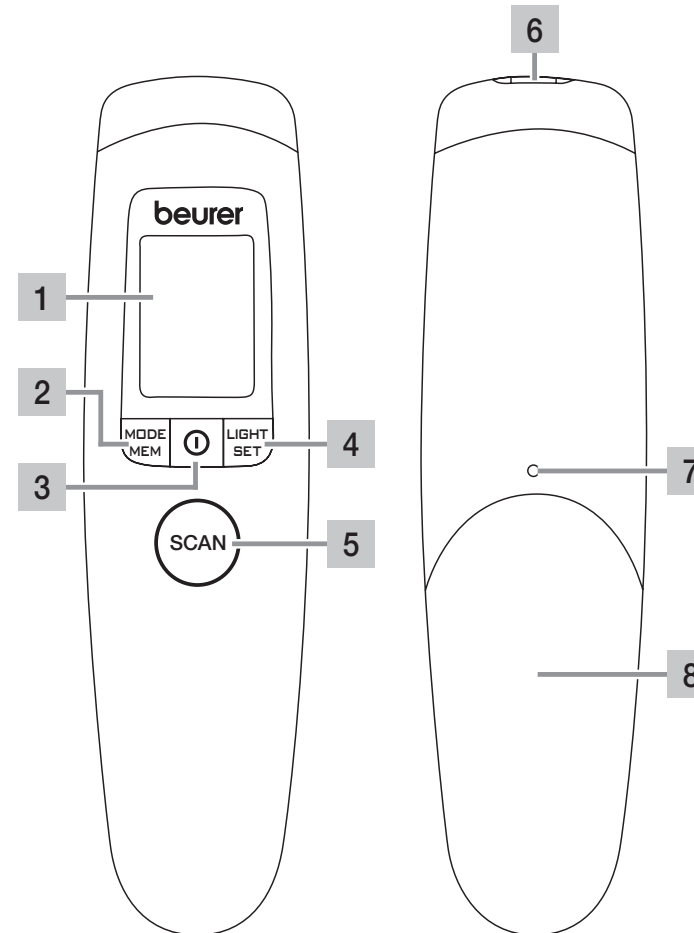
Beurer tip

Taking the body temperature provides a current measurement of a person's temperature. If you are uncertain about interpreting the results or if the values are abnormal (e.g. fever), please consult your doctor. This also applies in the case of slight temperature changes if there are other symptoms of illness such as agitation, severe sweating, flushed skin, fast pulse rate, tendency to collapse, etc.

3. Unit description


- 1 Display
- 2 **MODE/MEM** button
- 3 On/Off button **ⓘ**
- 4 **LIGHT/SET** button
- 5 **SCAN** button
- 6 Measuring sensor
- 7 Battery compartment lock
- 8 Battery compartment lid

Buttons	Functions
ⓘ	Turns the device on and off.
SCAN	Starts the temperature measurement.
MODE/MEM	MODE Sets the measurement mode. MEM Displays stored measurements.
LIGHT/SET	LIGHT Manual switch on of the illuminated display. SET Sets the basic functions.

3.1 Display description

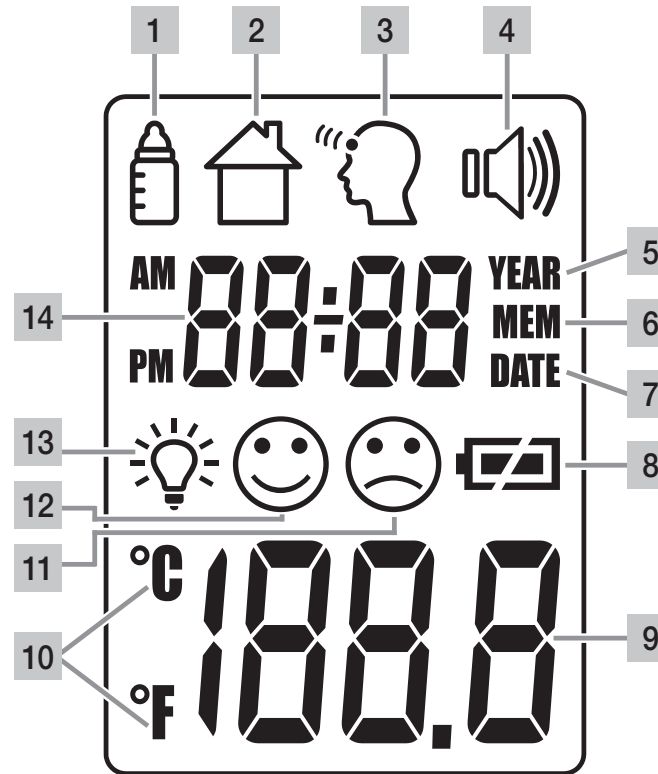- 1 Object temperature mode
- 2 Room temperature mode
- 3 Forehead temperature mode
- 4 Acoustic signal symbol
- 5 Year
- 6 Memory function
- 7 Date
- 8 Battery level
- 9 Temperature/memory space number display
- 10 Celsius/Fahrenheit measurement unit
- 11 Measurement result: "fever"
- 12 Measurement result: "no fever"
- 13 Illuminated display symbol
- 14 Year/date/time display

4. Initial use

If provided, pull out the battery insulating strip at the battery compartment or remove the protective film from the battery and insert the battery with the correct polarity.

▷ 9. Batteries

After a brief self-test and two short beeps the thermometer is ready for forehead measurement.

5. Switching on and setting the thermometer

Briefly press the On/Off button **ⓘ**.

After a brief self-test and two short beeps the thermometer is ready for forehead measurement.

The device always starts up in forehead temperature mode .

5.1 Setting the basic functions

This menu allows you to set the following functions individually, one after another.

Time format

- With the thermometer switched on, press and hold the **LIGHT/SET** button for 5 seconds. The time format option flashes on the display (fig. 1).
- Use the **MODE/MEM** button to set your preferred time format and confirm with the **LIGHT/SET** button.

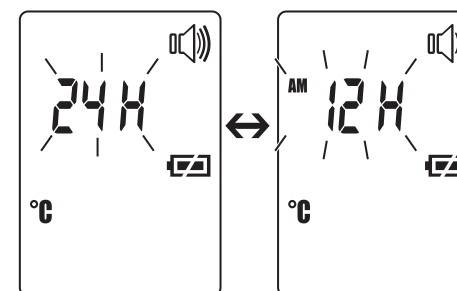
Fig. 1

Date

- The year flashes in the display (fig. 2).
- Use the **MODE/MEM** button to set the year and confirm with the **LIGHT/SET** button.
- Day/month flashes in the display (fig. 3).
- Use the **MODE/MEM** button to set the day and month, and confirm with the **LIGHT/SET** button.
- ⓘ** In the 24h format, the date is displayed as day/month. In the 12h format, it is displayed as month/day.

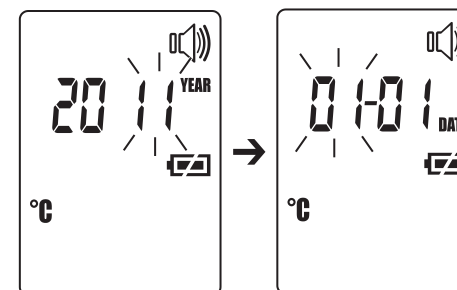
Fig. 2

Fig. 3

The time flashes in the display (fig. 4).

- Use the **MODE/MEM** button to set the time and confirm with the **LIGHT/SET** button.

① In the 12h format, the time is displayed as AM/PM.

Fig. 4

The temperature measurement unit flashes in the display (fig. 5).

You can set the device to display the temperature in degrees Celsius (°C) or degrees Fahrenheit (°F).

- To display the temperature in Celsius, select **°C** using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.
- To display the temperature in Fahrenheit, select **°F** using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.

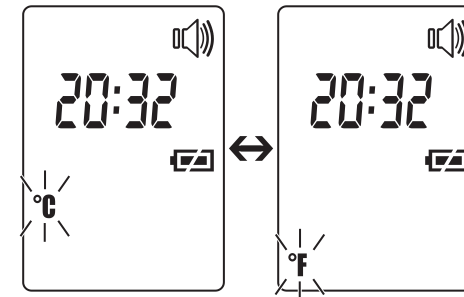
Fig. 5

The acoustic signal symbol flashes in the display (fig. 6).

You can activate/deactivate the acoustic signals (device activation, measurement in progress, measurement completion).

- To activate the acoustic signals, select **ON** using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.
- To deactivate the acoustic signals, select **OFF** using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.

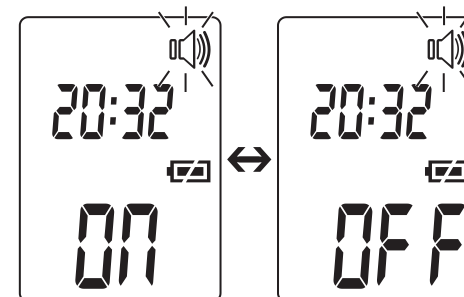
Fig. 6

The light symbol flashes in the display (fig. 7).

You can activate or deactivate the automatic illuminated display (appears following forehead temperature measurement).

- To activate the automatic illuminated display, select **ON** using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.
- To deactivate the automatic illuminated display, select **OFF** using the **MODE/MEM** button and confirm with the **LIGHT/SET** button.

You can also activate the illuminated display manually by briefly pressing the **LIGHT/SET** button.

The display lights up for 5 seconds.

- ❗ The automatic and manual illuminated display cannot be used while measurement is in progress.

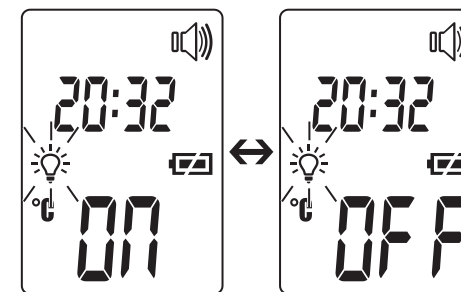
Fig. 7

6. Forehead measurement

Briefly press the On/Off button ①.

After a brief self-test and two short beeps the device is ready for forehead temperature measurement. The device is in forehead temperature mode. This is indicated by the "head" symbol.

Beurer tip

Remember that

- physical activity, increased perspiration on the forehead, taking vasoconstrictive medication and skin irritations can distort the result,
- the forehead, or the temples, must be free from perspiration and cosmetics.

Hold the thermometer 2 to 3 cm from the measuring point. Press the **SCAN** button and move the thermometer from side to side over the forehead area (fig. 8).

During measurement you will hear short beeps (only if the acoustic signal is activated), which signal that the thermometer has found a new highest measurement.

End of measurement is signalled by a long beep.

Release the **SCAN** button.

You can now read the measured value.

In addition to the temperature, the fever ☹️ or no fever 😊 symbols also appear in the display:

The no fever symbol 😊 indicates that the body temperature is within normal range; the fever symbol ☹️ indicates a measurement of over 38 °C, i.e. temperature alarm.

The measuring time is usually 2 seconds but may take up to 30 seconds.

The device is ready to take another measurement when two short beeps sound and the forehead symbol 🗑️ stops flashing.

The measurement is automatically saved with the date/time and the “fever” ☹️/ “no fever” 😊 classification.

6.1 Displaying saved measurements

The device only stores measurements in forehead temperature mode 🗑️. The device automatically stores the values from the last 60 measurements.

When 60 storage places are exceeded the oldest value is deleted.

The memory can be called up as follows:

- With the thermometer switched on, press and hold the **MODE/MEM** button for 5 seconds.
The most recent measurement is displayed.
- With each subsequent pressing of the **MODE/MEM** button, first the storage space number is displayed followed by the measurement upon release.
- In the upper line, the time and date are displayed alternately.

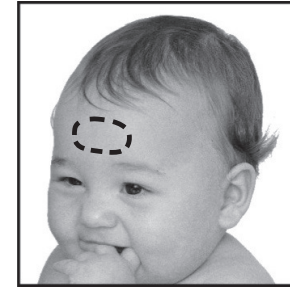
Fig. 8

7. Measuring object temperature/room temperature

Object temperature mode

If you want to measure an object temperature with this thermometer, switch to object temperature mode.

- With the thermometer switched on, briefly press the **MODE/MEM** button.

The device switches to the object temperature mode .

- Hold the thermometer 2 to 3 cm from the measuring point. Briefly press the **SCAN** button and read the temperature in the display (fig. 9).

Measurements taken in object temperature mode are not stored.

Room temperature mode

If you want to use the thermometer to measure the room temperature you need to switch to room temperature mode.

- With the thermometer switched on, briefly press the **MODE/MEM** button twice.

The device switches to the room temperature mode .

- The room temperature is immediately displayed (fig. 10).

Measurements taken in room temperature mode are not stored.

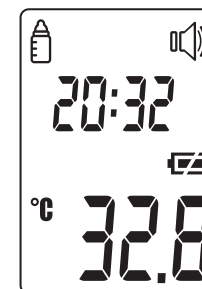
Fig. 9

Fig. 10

8. Error messages


Error message	Problem	Solution
	Measurement during self-test, device not yet ready for measurement.	Wait until the forehead symbol stops flashing.

Error message	Problem	Solution
	Large variation in room temperature.	Store the device for at least 30 minutes in a room with room temperature (10–40 °C/50–104 °F).
	Room temperature below 10 °C or over 40 °C (<50 °F, >104 °F).	Room temperature must be between 10 °C and 40 °C (<50 °F, >104 °F).
	Faultless functionality cannot be guaranteed.	Remove the battery for approx. 1 minute and then reinsert. In the event of a repeated error message, contact your retailer or Customer Services.
	(1) Forehead temperature mode: The temperature recorded is higher than 42.2 °C (108 °F). (2) Object temperature mode: The temperature recorded is higher than 80 °C (176 °F).	Operate the thermometer only between the specified temperature ranges. In the event of a repeated error message, contact your retailer or Customer Services.
	(1) Forehead temperature mode: The temperature recorded is lower than 34 °C (93.2 °F). (2) Object temperature mode: The temperature recorded is lower than -22 °C (-7.6 °F).	Operate the thermometer only between the specified temperature ranges. In the event of a repeated error message, contact your retailer or Customer Services.
	Self-test unsuccessful.	Replace the batteries.
	The batteries are empty.	Replace the batteries.

9. Batteries

The device requires two AAA (LR03) batteries.

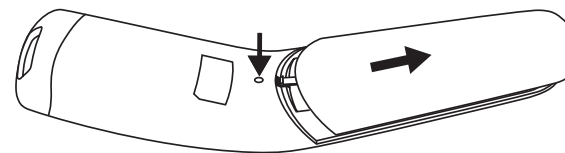- Open the battery compartment.
Use a pointed object to press down on the battery compartment lock and at the same time slide the battery compartment downwards.
- Take the used batteries out of the battery compartment.
- Insert new batteries.
Make sure that the batteries are inserted the right way round.
- Close the battery compartment.

Used batteries should not be disposed of in normal household waste. You are legally required to dispose of the batteries. Dispose of them via your electronics retailer or your local recycling point.

Note: The codes below are printed on batteries containing harmful substances: Pb = battery contains lead, Cd = battery contains cadmium, Hg = battery contains mercury. The batteries in this device do not contain any pollutants.

10. Cleaning the device

- The measuring sensor is the most sensitive part of the thermometer. Take great care of the measuring sensor when cleaning the device.
- Do not use any harsh cleaning products.
- Always observe all safety notes for user and device.
Safety notes ▷ Page 19.

Clean the measuring sensor after each use. Use a clean cloth or cotton bud that can be moistened with disinfectant or 70 % alcohol.

To clean the entire device, please use a soft cloth slightly moistened with a mild soapy solution. Under no circumstances may liquid enter the device.

Do not use the device again until it is completely dry.

11. Storing the device

The device must not be stored or used at an excessively high or low temperature or humidity (see technical specifications), in sunlight, in association with an electrical current or in dusty locations. Otherwise inaccuracies can occur.

If prolonged storage is planned, you should remove the batteries.

12. Disposing of the device

Dispose of the device in accordance with EC Directive 2002/96/EC – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please contact the appropriate local authorities.

13. Technical Data

If the device is used other than in accordance with the specifications, perfect functioning cannot be guaranteed!

We reserve the right to make technical changes to improve and develop the product.

Measurement method	Non-contact infrared measurement
Basic functions	Forehead temperature measurement Object temperature measurement Room temperature measurement
Measurement units	Celsius (°C) and Fahrenheit (°F)
Operating conditions	10 °C to 40 °C (50 °F to 104 °F) with a relative humidity of < 95 %
Storage conditions	-20 °C to 50 °C (-4 °F to 122 °F) with a relative humidity of < 85 %
Measurement distance	2 to 3 cm from the measuring point

Measurement range and accuracy of forehead temperature measurement	Forehead temperature measurement 34 °C to 42.2 °C (93.2 °F to 108 °F) Measurement accuracy 36 °C to 39 °C: ±0.2 °C (96.8 °F to 102 °F: ±0.4 °F) In the remaining measurement range ±0.3 °C (±0.5 °F)
Clinical repeat precision	0.23 °C (0.41 °F)
Measurement range and accuracy Object temperature measurement	Object temperature measurement -22 °C to 80 °C (-7.6 °F to 176 °F) Measurement accuracy ±4% or ±2 °C (±4 °F)
Memory function Body temperature measurement	Automatically stores the last 60 measurements.
Acoustic signal	With device activation, during measurement, when measurement is complete (can be set).
Display	LCD display
Energy saving functions	Device automatically switches off after 1 minute.
Dimensions Width x depth x height	approx. 47.6 mm x 29.0 mm x 188.0 mm
Weight	82 g (without batteries)
2 x AAA (LR03) batteries	Batteries last for approx. 3000 measurements Activated functions such as acoustic signal or illuminated display also reduce the battery life.
Signs and symbols	 Device classification type BF  Please read the instructions for use.

14. Guidelines

- This device complies with the EU Directive 93/42 EEC.
- This device complies with ASTM E 1965-98.
- This device complies with European Standard EN60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility.

15. Included in delivery

- Clinical thermometers
- 2 x AAA (LR03) batteries
- Storage box
- Instructions for use

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